

# INSTALLATION & USER MANUAL

(FormUS™  
Positech innovation

(FlexUS™  
Positech innovation

(BackUS™  
Positech innovation

(CurvUS™  
Positech innovation

## 1.0 TABLE OF CONTENTS

	Page
<b>1.0 Table of contents</b>	<b>2</b>
<b>2.0 Important considerations</b>	<b>3</b>
<b>3.0 User, clinician &amp; dealer satisfaction</b>	<b>4</b>
<b>4.0 Warranty &amp; limitation of liability</b>	<b>5</b>
<b>5.0 Installation and adjustment with FLEXUS</b>	<b>6</b>
<b>6.0 Installation and adjustment with BACKUS &amp; CURVUS</b>	<b>7</b>
<b>7.0 Cleaning care instructions</b>	<b>7</b>



## 2.0 IMPORTANT CONSIDERATIONS



Positech innovation Inc. accepts no liability for personal injury or damage to property that may arise from the failure of the user or other persons to follow the recommendations, warnings, and instructions in this Installation & Care Manual.



All persons responsible for fitting, adjustment, and daily use of the devices discussed in these instructions must be familiar with and understand all safety aspects of the devices mentioned. In order for our products to be used successfully, you must :

- Read and understand all instructions and warnings.
- Maintain our products according to our instructions on care and maintenance.
- Devices should be installed and adjusted by a trained qualified technician or clinician.



Check all parts for shipping damages before using. In case of damage, do NOT use the equipment. Contact Positech innovation Inc. for further instructions.



DO NOT install this equipment without first reading and understanding this instruction booklet. If you are unable to understand these instructions, contact a clinician, vendor or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur.



**DEALER/CLINICIAN :** The **User Manual** must be given to the consumer of any of Positech innovation Inc. products.

**DEALER/CLINICIAN :**



- Inform the User to immediately discontinue use if any function is compromised, parts are missing, loose, or shows signs of excessive wear. Consult with Positech innovation Inc. for repair, adjustment, or replacement.
- A faulty usage of this product could worsen the user's condition instead of improving it.
- Usage of this product for therapeutic purposes such as posture correction and support must be conducted in collaboration with a therapist.
- A therapist must absolutely supervise usage of this product in the case of severe deformities, such as scoliosis.



**DEALER/CLINICIAN :** Monthly verifications are necessary to ensure that :

- Components are fastened securely.
- Adjustments remain adapted to the patient's condition evolution. Further adjustments may be necessary in the event of a weight gain or loss, or an improvement or a worsening of the posture. If in doubt, please refer to a specialist.



**USER/CAREGIVER :** Before using this product, read the entire **User Manual** and save for future reference. It is important to refer to the **User Manual** for additional safety information related to the safe operation of all our products.



This **Installation & Care Manual** is compiled from the latest specifications and product information available at the time of publication. We reserve the right to make changes as they become necessary. Any changes to our products may cause slight variations between the illustrations and explanations in this manual and the product you have purchased. Information contained within this document is subject to change without notice.



It is very important that you read and understand all instructions prior to the installation & care of these products. Failure to adhere to instructions and warnings in this document may result in property damage, injury, or death. Product misuse due to failure of the following instructions will void the warranty.



You can download additional copies of this Installation & Care Manual on the Positech innovation Inc. website.

## 3.0 USER, CLINICIAN & DEALER SATISFACTION

# You sit well with us.

Positech innovation Inc. strives for 100% customer satisfaction.

Your complete satisfaction is important to us.

Please contact us with feedback or suggestions that will enhance the quality and functionality of these products.

We can be reached at :

<b>Mail</b>	<b>100-1566 Rue Nationale, Terrebonne, Québec, Canada, J6W 0E2</b>
<b>Email</b>	<b>service@positechinnovation.com</b>
<b>Website</b>	<b>www.positechinnovation.com</b>
<b>Phone</b>	<b>(1) 450 964 3232</b>
<b>Toll Free</b>	<b>(1) 877 964 3232</b>
<b>Fax</b>	<b>(1) 450 964 3378</b>
<b>Customer Service Hours</b>	<b>Monday through Friday – 8h00 to 17h00 Eastern Standard Time ( EST )</b>

### FOR YOUR RECORD

<b>INSTALLATION DATE</b>		<b>CLINIC/HEALTHCARE CENTRE/PRACTICE</b>	
<b>USER INFORMATION</b>		<b>DEALER STAMP</b>	
<b>NAME:</b>			
<b>EMAIL/PHONE:</b>			
<b>FILE #:</b>		<b>SERIAL NUMBER (if applicable)</b>	

## 4.0 WARRANTY & LIMITATION OF LIABILITY

Metallic components of products purchased from Positech are warranted against defects in workmanship and material for a period of two (2) years from the date of delivery of the product to the first consumer.

Textile components of products purchased from Positech are warranted against defects in workmanship and material for a period of ninety (90) days from the date of delivery of the product to the first consumer.

Non-metallic and non-textile components of products purchased from Positech are warranted against defects in workmanship and material for a period of one (1) year from the date of delivery of the product to the first consumer.

Positech's obligations under this warranty are limited to replacing (transportation costs at the Dealer's expense) or repairing, at its option, any products which under normal and proper use and maintenance are proved to be defective to Positech, in its sole discretion, within the warranty period, provided that Positech has had the opportunity to inspect them.

This warranty is not transferable and only applies to the first consumer purchase through an authorized Positech dealer.

This warranty does not cover products or parts thereof which (i) normally wear out due to the general wear and tear of standard use, (ii) are modified, altered, inadequately stored or kept, or (iii) are not installed, adjusted, used or maintained in accordance with Positech's instructions. In addition to the foregoing, this warranty does not cover punctures, tears, or burns, exposure to moisture, exceeding the weight limit and <acts of nature> ( ex. Flood, Earthquake, Tornado, etc. ). Any substitutions of supplied hardware also void the warranty.

THE FOREGOING IS POSITECH'S SOLE WARRANTY, ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OF FITNESS FOR PURPOSE AND MERCHANTABILITY, ARE EXCLUDED AND DISCLAIMED TO THE FULL EXTENT PERMITTED BY LAW. IN NO EVENT SHALL THE DEALER BE ENTITLED TO MORE THAN THE PRICE OF THE PURCHASED PRODUCTS AT THAT TIME AS FINAL PENALTY AND DAMAGE. IN NO EVENT, WHETHER AS A RESULT OF CONTRACT, TORT, STRICT LIABILITY OR OTHERWISE, SHALL POSITECH BE LIABLE FOR SPECIAL, INCIDENTAL, CONSEQUENTIAL OR PUNITIVE DAMAGES, INCLUDING BUT NOT LIMITED TO, LOSS OF USE, PROFIT OR REVENUE.

The Dealer understands and agrees that the Positech's products may be dangerous and result in injury, worsening of the condition or damage to property if not used properly. Consequently, the Dealer shall inform the user that he/she must (i) comply with all specifications, instructions and other requirements provided by Positech with the products, on Positech's website, in manuals, data sheets, on labels or packaging, and the terms & conditions uniting parties before making use of the products, and (ii) use the products under the supervision of a therapist in the case of severe deformities or when used for therapeutic purposes.

The Dealer shall hold harmless, indemnify and defend Positech from and against any and all claims, demands, causes of action, suits or judgments (including costs, expenses and reasonable attorney fees incurred in connection therewith) for the injury to any person or for loss of or damage to property arising out of or in connection with the improper use of the products whether or not caused by Dealer's negligence and for any default hereunder.


The Dealer may have additional rights by law which may vary from one jurisdiction to another. Positech does not hereby intend to limit or exclude the rights of the Dealer beyond the limits allowed by law.

This warranty applies in CANADA only.

### WARRANTY SERVICE

To obtain service under this warranty, the warranty-eligible Product must be presented to the nearest Positech innovation Inc. authorized dealer. In Canada, warranty service may also be obtained by calling (877)964 3232, Monday through Friday 8h00 to 17h00 Eastern Standard Time. Owner is responsible for all costs associated with returning Product to Positech innovation Inc. for service. Where at its sole discretion Positech innovation Inc. deems Product eligible for warranty repair or replacement, Positech innovation Inc. will pay the shipping and handling associated with returning repaired or replaced Product to Owner. In instances where Positech innovation Inc. deems a returned product ineligible for warranty service, Positech innovation Inc. will, if possible, repair product for a reasonable charge that includes return shipping and handling.

## 5.0 INSTALLATION AND ADJUSTMENT with FLEXUS

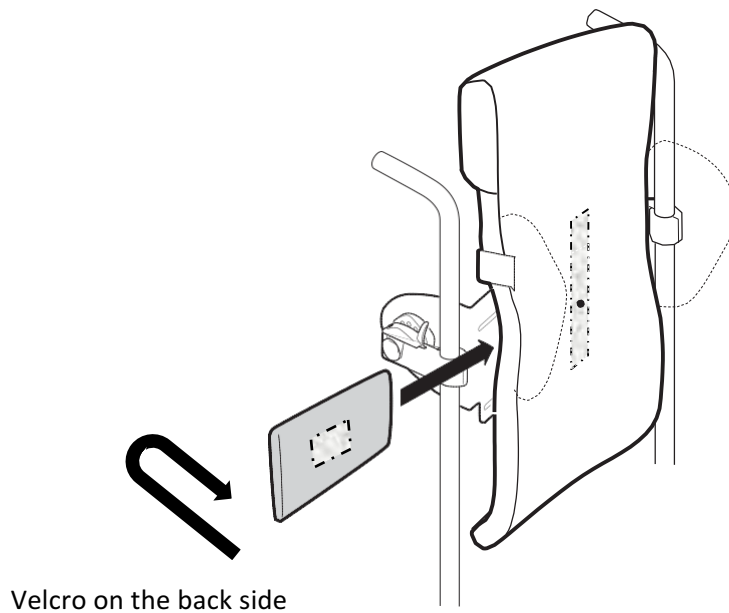

Select the desired size & profile of FORMUS.


FORMUS is installed in between the rigid structure of the backrest and the upholstered backrest cushion. It is secured by Velcro.

A Velcro strip is located onto the interior side of lower mobile plate. It will connect with the Velcro strip located on the FORMUS.


FROM THE SIDE, insert at the desired height FORMUS making sure that its VELCRO strip is facing the interior side of the structure.

Connect both Velcro surfaces at the desired height to optimize the profile of the backrest.

## 6.0 INSTALLATION AND ADJUSTMENT with BACKUS & CURVUS

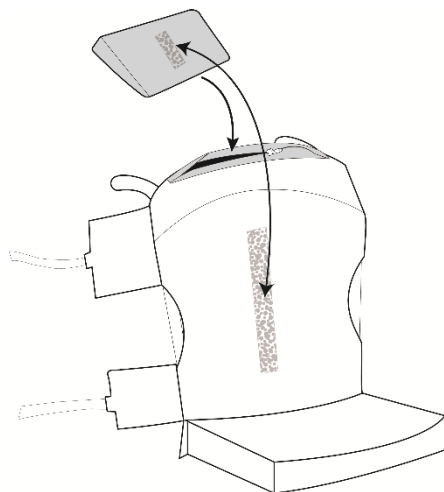

**BACKUS**

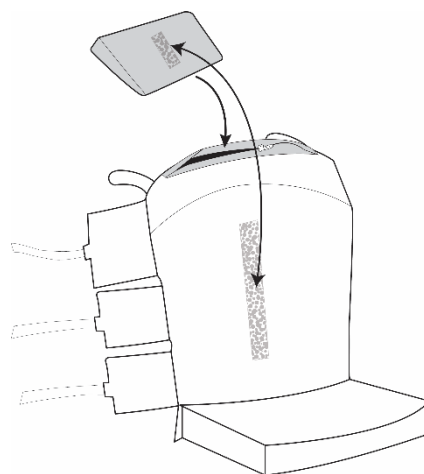

**CURVUS**

Select the desired size & profile of FORMUS.

Unzip the upper part of the upholstered backrest cushion.

On the inside of the frontal surface of the cover, there is a Velcro strip.

Insert in the desired FORMUS while make sure that its Velcro is facing the front of the wheelchair.

Connect both Velcro surfaces at the desired height to optimize the profile of the backrest.

## 7.0 CLEANING CARE INSTRUCTIONS



**DO NOT IMMERSE COMPLETELY.**

Wipe with a damp cloth and dry flat.



#### CORPORATE HEADQUARTERS

100-1566 Rue Nationale,  
Terrebonne, Québec, Canada,  
J6W 0E2

TEL: 1 450-964-6262

1 877-964-3232

FAX: 1 450-964-3378

[www.positechinnovation.com](http://www.positechinnovation.com)  
[service@positechinnovation.com](mailto:service@positechinnovation.com)